

PhotoMed SDL - Additional settings for iPhone 15 Pro/16 Pro/17 Pro

Apple has designed the iPhone Pro models to detect when you are taking a closeup photo and then automatically use the “macro” mode. This mode is not ideal for clinical photos. It changes the camera to the wide angle camera which introduces spherical distortion into the photo. There is a way to prevent the phone from using the macro mode. Follow the steps below to keep using the telephoto camera for all of your clinical photos.

Tap the Settings icon on your phone. Scroll down and tap on Apps (image #1 below) and then Photos (image #2 below). Turn off View Full HDR (image #3 below).

Go back to Settings and tap on Camera (image #4 below). Tap on Preserve Settings (image #5 below), then turn on Macro Control and Live Photo (image #6 below).

Go to the camera app. Whenever you take clinical photos, always start by tapping the 3X icon. Move the phone near a subject until the yellow flower icon appears in the lower right corner (image #7 below). Tap the yellow flower icon to turn off macro mode. Tap the “Six Dots” icon in the upper left corner to open additional settings - turn off Live Photos (image #8 below).

The phone will now remember these setting choices. Image #9 shows the settings as having been turned off. If the yellow flower icon re-appears, be sure to tap it to turn off Macro Mode before taking photos.

